



## **Mix2Vial™ Transfer Device**

Please refer to the device instructions for use (IFU) for complete set of directions for the device.

### **Intended Use/Indications for Use:**

The Mix2Vial™ transfer device is intended for transferring and mixing drugs contained in two vials.

### **Contraindications:**

Currently no specific contraindications are known for the Mix2Vial™ transfer device.

### **Cautions/Precautions/Warnings:**

- Federal (USA) law restricts this device to sale by or on the order of a physician.
- This device is intended to be used for one patient and one application only.
- Contents are sterile and non-pyrogenic.
- Reuse compromises safety and efficacy – it may cause contamination due to loss of sterility.
- Re-sterilization may damage the device
- A Do not remove the device from vials after use.
- DO NOT use if package is damaged.
- Dispose of used device in accordance with applicable regulations.

### **Registration Information**

Information herein is for use only in countries with applicable health authority registrations. The Mix2Vial™ Transfer device is 510(k) cleared by the United States Food and Drug Administration and carries the CE mark (0344). Products are shown for INFORMATION purposes only and may not be approved for marketing in specific regions. Please contact your West Pharmaceutical Services, inc. (West) representative for product availability.

*West and the diamond logo are registered trademarks of West Pharmaceutical Services, Inc. in the U.S. and other jurisdictions.*

**Date:** 06-February-2022

**Number:** GN-REG-240

**Title:** Mix2Vial™ Transfer Device - Indications, Safety and Registration Zones

**Revision:** 1.0