



VIAL ADAPTER™ TRANSFER DEVICES

A solution for single-use transfer of drugs in vials.



The Vial Adapter™ transfer device is a single-use, sterile, medical device intended for the transfer of drugs contained in vials. Puncturing the elastomeric closure of a vial is achieved by means of an integral plastic spike located in the center of the vial adapter. The device plastic package and cover are designed to limit direct contact, reduce potential for contamination, and to assist in attaching the device to the drug vial.



VIAL ADAPTER™ TRANSFER DEVICES

- Options for siliconized spike and in-line particulate filters (available in 5µm, 15µm and 25µm pore sizes)

VENTED VIAL ADAPTER™ TRANSFER DEVICES

- One piercing spike with two lumens:
 - One for transfer of drug/solution between the drug vial and a syringe
 - Second for enabling pressure equilibrium of the vial by introducing air that has passed through a 0.2 micron hydrophobic air filter into the vial.

SWABABLE VIAL ADAPTER™ TRANSFER DEVICES

- Consists of the vial adapter body with a single lumen piercing spike and a swabable valve which allows multiple needle-free accesses to drug vial to facilitate withdrawal of medication for treatment.

Rx Only.

Vial Adapter™, Vented Vial Adapter™, and Swabable Vial Adapter™ transfer devices carry the CE mark (0344) and are 510(k) cleared by the United States Food and Drug Administration. Products are shown for INFORMATION purposes only and may not be approved for marketing in specific regions. Distribution and use are subject to applicable regulatory approvals and requirements for medical devices. The Vial Adapter™ and Vented Vial Adapter™ transfer devices are configurable and may not be suitable for use with all drugs. The Swabable Vial Adapter™ transfer device inhibits microbial ingress for up to 24 hours and 72 hours, respectively, when used in accordance with the product instructions for use and proper aseptic technique. The use of the Swabable Vial Adapter™ should not be interpreted as modifying, extending, or superseding a drug manufacturer's labeling and use instructions for device configuration compatibility and for shelf life and sterility recommendations for storage and expiration dating. The Swabable Vial Adapter™ transfer device is not compatible for use with all drug products. Refer to drug manufacturer's labeling and use instructions for device configuration compatibility and for shelf life and sterility information of opened or reconstituted drug product. Failure to follow product instructions for use may result in compromised sterility; contamination; leakage (including possible exposure to medication); and/or inadequate medication reconstitution, transfer, and/or dosing. Product misuse could potentially lead to user and/or patient infection, suboptimal therapy, and/or delay in therapy. Please contact your West Pharmaceutical Services, Inc. (West) representative for product availability. Important product and safety information and warnings at: <https://www.westpharma.com/products/vial-adapter-systems/vial-adapters>

FOR MORE INFORMATION VISIT **WESTPHARMA.COM**

NORTH AMERICA 1-800-345-9800 OPTION 8 | **SOUTH AMERICA** +55 11 4055 6061

EUROPE +49 (0) 2403-7960 | **ASIA PACIFIC** +65 6862 3400

WEST PHARMACEUTICAL SERVICES, INC. 530 HERMAN O. WEST DR. | EXTON, PA 19341

West and the diamond logo and By your side for a healthier world™ are trademarks or registered trademarks of West Pharmaceutical Services, Inc. in the United States and other jurisdictions. Vial Adapter™, Vented Vial Adapter™, and Swabable Vial Adapter™ are trademarks of West Pharma. Services IL, Ltd., a subsidiary of West Pharmaceutical Services, Inc.

Copyright © 2024. West Pharmaceutical Services, Inc.

#11758 • 0324